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Artwork consists of:

- One 4 x 20-inch graphic attached.

Artwork master disk contains the following file(s):

File Name	Description
AW-24896-001_001_01.zip	Source and Supplier print file
AW-24896-001_001_02.pdf	View File


Artwork prints black and Pantone 186 (red)

REV AUTHORED BY G. NIKHIL	DATE JULY/23/2021	<b>HOLOGIC</b> <sup>®</sup>		SIGNATURES ON FILE
REV DRAFTED BY G. NIKHIL	DATE JULY/23/2021			
PROPRIETARY: This document contains proprietary data of Hologic, Inc. No disclosure, reproduction or use of any part thereof may be made except by written permission from Hologic.		TITLE <b>IFU, THINPREP STAIN EA SOLUTION, IVDR, HARD COPY, EN</b>	DOCUMENT NUMBER <b>AW-24896-001</b>	REV 001
REV. RELEASE DATE	4/1/2022	SIZE A	SHEET 1 OF 1	

Before using this document, please consult Agile for the latest revision.

Form ENG-0034-T01, Rev. 006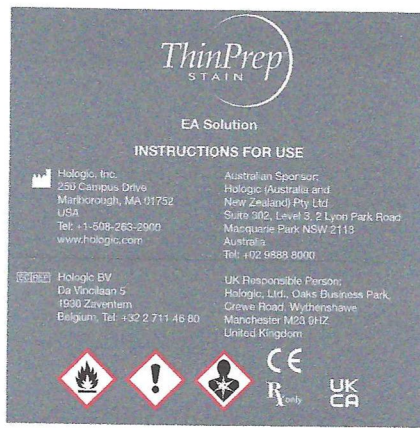

**INTENDED USE**

ThinPrep™ Stain EA Solution is intended to be used in a Papanicolaou staining procedure in conjunction with any ThinPrep Stain Nuclear Stain, ThinPrep Stain Rinse Solution, ThinPrep Stain Bluing Solution, and ThinPrep Stain Orange G Solution. For professional use.

**SUMMARY AND EXPLANATION**

ThinPrep EA Solution is an alcoholic solution that serves to stain the cytoplasm of cells on slide preparations for cytologic evaluation.

**PRINCIPLES OF PROCEDURE**

When used in conjunction with the ThinPrep staining solutions and cytologic slide preparations, ThinPrep EA Solution will stain the cytoplasm of cells without interfering with nuclear staining and morphology.

**COMPOSITION**

An alcoholic solution of Ethanol, Isopropyl Alcohol, Methanol, and proprietary biological dyes. CAS-64-17-5, CAS 67-56-1, CAS 67-63-0

**WARNINGS**

**Danger.** Flammable. Contains Ethanol, Isopropyl Alcohol, Methanol.  
 H302. Harmful if swallowed.  
 H312. Harmful in contact with skin.  
 H319. Causes serious eye irritation.  
 H371. May cause damage to organs.  
 H225. Highly flammable liquid and vapor.  
 For In Vitro Diagnostic Use. Not for external or internal use in humans or animals. Cannot be made non-poisonous.

**PRECAUTIONS**

P210. Keep away from heat/sparks/open flames/hot surfaces. — No smoking.  
 P280. Wear protective gloves and eye/face protection.  
 P280. Do not breathe dust/fume/gas/mist/vapours/spray.  
 P301 + P310. IF SWALLOWED: Immediately call a POISON CENTER or doctor/physician.  
 P303 + P361 + P533. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.  
 P305 + P351 + P338. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.  
 P304 + P340. IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.  
 P309 + P311. IF exposed or if you feel unwell: Call a POISON CENTER or doctor/physician.  
 P370 + P378. In case of fire: Use dry sodium carbonate to extinguish.  
 P403 + P233. Store in a well-ventilated place. Keep container tightly closed.  
 P501. Dispose of contents/container in accordance with local/regional/national/international regulation.  
 Wash hands thoroughly after handling. Keep container tightly closed when not in direct use. Use with adequate ventilation.  
 Results with non-ThinPrep cytology samples have not been evaluated. Do not use if primary packaging is damaged.

**PRETREATMENT**

No reconstitution, mixing or dilution is required.

**STORAGE**

Store ThinPrep EA Solution at 15°C (59°F) to 30°C (86°F) in the container provided. Do not use ThinPrep EA Solution beyond the expiration date marked on the container.

**APPEARANCE AND INTEGRITY**

Opaque, non-sterile solution.

**SPECIMEN COLLECTION AND PREPARATION**

For use with gynecologic cytology samples that have been processed on any ThinPrep™ Processor.

**PROCEDURE**

See the ThinPrep Stain User's Manual for specific staining protocols.

**LIMITATIONS OF PROCEDURE**

Must be used according to the instructions provided in the ThinPrep Stain User's Manual. ThinPrep EA Solution cannot be substituted with any other solution.

**SAFETY AND PERFORMANCE CHARACTERISTICS**

When used as directed, ThinPrep EA Solution will stain the cytoplasm of cells on cytologic slide preparations.

If any serious incident occurs related to this device or any components used with this device, report it to Hologic Technical Support and the competent authority local to the patient and/or user.

Refer to the ThinPrep Imaging System operator's manual.

**DISPOSAL**

Dispose in accordance with all applicable regulations.

**FIRST AID MEASURES**

IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell.

See [www.hologicds.com](http://www.hologicds.com) for the entire Safety Data Sheet.

**Revision History**

Revision	Date	Description
AW-24896-001 Rev. 001	3-2022	Clarify instructions. Add instructions regarding reporting serious incidents.