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Artwork consists of:

- One 4 x 20-inch graphic attached.

Artwork master disk contains the following file(s):

File Name	Description
AW-24890-001_001_01.zip	Source and Supplier print file
AW-24890-001_001_02.pdf	View file


Artwork prints black and Pantone 186 (red)

REV AUTHORED BY M. HARUN	DATE JULY/27/2021	HOLOGIC®		SIGNATURES ON FILE
REV DRAFTED BY M. HARUN	DATE JULY/27/2021			
PROPRIETARY: This document contains proprietary data of Hologic, Inc. No disclosure, reproduction or use of any part thereof may be made except by written permission from Hologic.		TITLE IFU, THINPREP STAIN RINSE SOLUTION, IVDR, HARD COPY, EN	DOCUMENT NUMBER AW-24890-001	REV 001
REV RELEASE DATE 4/1/2022		SIZE A	SHEET 1 OF 1	

Before using this document, please consult Agile for the latest revision.

Form ENG-0034-T01, Rev. 006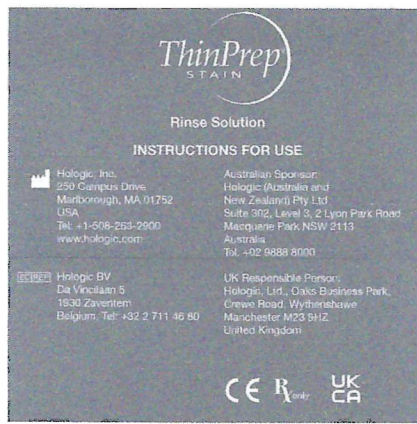

INTENDED USE

ThinPrep™ Stain Rinse Solution is intended to be used in a Papanicolaou staining procedure in conjunction with ThinPrep Stain Nuclear Stain, ThinPrep Stain Bluing Solution, ThinPrep Stain Orange G Solution, and ThinPrep Stain EA Solution. For professional use.

SUMMARY AND EXPLANATION

ThinPrep Rinse Solution is an aqueous, detergent solution that removes excess ThinPrep Nuclear Stain from cellular components on slide preparations for cytologic evaluation.

PRINCIPLES OF PROCEDURE

When used in conjunction with the ThinPrep staining solutions and cytologic slide preparations, ThinPrep Rinse Solution will remove excess ThinPrep Nuclear Stain from cellular components, enhancing nuclear staining and clarifying cytoplasm.

COMPOSITION

An aqueous solution of propriety surfactant.

WARNINGS

For In Vitro Diagnostic Use.

Not for external or internal use in humans or animals. May cause irritation on contact.

PRECAUTIONS

Avoid contact with eyes.

Wear gloves and eye protection when handling solution.

Keep container closed when not in direct use.

Results with non-ThinPrep cytology samples have not been evaluated.

Do not use if primary packaging is damaged.

PRETREATMENT

No reconstitution, mixing or dilution is required.

STORAGE

Store unused ThinPrep Rinse Solution at 15°C (59°F) to 30°C (86°F) in the container provided. Do not use ThinPrep Rinse Solution beyond the expiration date marked on the container.

APPEARANCE AND INTEGRITY

Clear, non-sterile solution.

SPECIMEN COLLECTION AND PREPARATION

For use with gynecologic cytology samples that have been processed on any ThinPrep™ Processor.

PROCEDURE

See the ThinPrep Stain User's Manual for specific staining protocols.

LIMITATIONS OF PROCEDURE

Must be used according to the instructions provided in the ThinPrep Stain User's Manual. ThinPrep Rinse Solution cannot be substituted with any other solution.

SAFETY AND PERFORMANCE CHARACTERISTICS

When used as directed on cytologic slide preparations, ThinPrep Rinse Solution removes excess ThinPrep Nuclear Stain from cellular components.

If any serious incident occurs related to this device or any components used with this device, report it to Hologic Technical Support and the competent authority local to the patient and/or user.

Refer to the ThinPrep Imaging System operator's manual.

DISPOSAL

Dispose in accordance with all applicable regulations.

FIRST AID MEASURES

No ill effects are anticipated. See www.hologicds.com for the entire Safety Data Sheet.

Revision History

Revision	Date	Description
AW-24690-001 Rev. 001	3-2022	Clarify instructions. Add instructions regarding reporting serious incidents.